

H2 O2 hydra facial machine

User manual



Dear customer, thanks for buying our product!

For the best capability and safety, please read this instruction manual before any operation.

Please reserve the instruction manual for future references.

1 Brief introduction:



Ultrasound

Ultrasonic vibration to quickly open the pores, allowing the essence to penetrate into the basal layer of the skin

Bipolar RF

Lifting and tightening, improving eye lines and promoting collagen re-generation



Skin Scrubber Vibration Cleaning

Remove dead skin by high-frequency vibration waves remove blackheads soften skin keratinocytes, and deeply cleanse skin dirt

Cold Hammer

Shrink pores, calm and repair damaged skin, eliminate sensitive skin



High Polymer Atomizing Pen

Combine with liquid products, the instrument atomizing the liquid product and quickly inject into the dermis.

Hydro Dermabrasion Cleaning

Remove blackheads remove grease and dirt, deep cleanse skin, brighten skin tone



Install according to the handle name in the picture and the name marked on the machine:



- ① Cold Jack
- ② RF Jack
- ③ Spray Jack



- ① Abrasion Jack
- ② Ultrasound Jack
- ③ Scrubber Jack



- ① Flow Adjust (hydro dermabrasion cleaning handle)
- ② Mode choose [A/B/C/D(H₂O₂) Bottle]
- ③ Vacuum intensity Adjust (hydro dermabrasion cleaning handle)
- ④ Disassembly button (A/B/C bottle)
- ⑤ A/B/C bottle
- ⑥ Power cable
- ⑦ Switch

(distilled water/pure water/Skin care products with cleansing effect after dilution)

water from A/B/C bottle clean skin by hydro dermabrasion cleaning handle then flow to waste bottle)



- ① Waste bottle
- ② H₂O₂ bottle (add distilled water/pure water , make water changed H₂, O₂, then flow to Spray gun)

2, Caution

a) Matters needing attention

*Precautions for use place

caveat:

- Please do not use parallel sockets to avoid fire;
- Be sure to turn off the power switch when replacing the fuse to avoid fire and electric shock;
- Using damaged or processed power cords may cause fire and electric shock;
- Don't bend or twist the power cord excessively to avoid fire and electric shock accidents;
- If the instrument is loose, dropped or damaged, please contact the company in time;
- Do not use the instrument in unstable environments such as tilt, vibration, or impact;
- No water or debris can enter the inside of the instrument. If foreign objects enter, please contact our company in time;
- The power plug must be unplugged when moving the instrument;
- When moving the instrument, avoid vibration as much as possible;
- The instrument should be placed in a stable place, and a space of at least 5 cm should be left around to ensure air circulation and facilitate heat dissipation of the instrument;
- Nothing should be covered on the instrument to prevent the vents from being blocked;
- Avoid using the instrument in the following environments: places exposed to sunlight; high humidity environments; near water environments; places subject to vibration and tilt; environments with strong magnetic fields; places where electromagnetic waves and impulse voltage occur; places where chemicals are stored are corrosive Gaseous places;

This instrument should not be placed and used near radios, televisions, copiers, fax machines, etc.;

Do not use with other instruments (such as microwave) to avoid mutual interference between the instruments and cause misoperation.

•Precautions before use

caveat:

- 1) Be sure to read the instruction manual carefully before starting up. The operator must be familiar with the instructions and methods of operation, and the instrument must be managed by a dedicated person;
- 2) Please set the operating parameters under the guidance of professionals;

note:

- 3) Confirm whether the instrument is in normal operation before use;
- 4) To confirm whether the ground wire is connected correctly;
- 5) To confirm whether all wires are correctly connected and safe;
- 6) When the instruments are used together, they must be used correctly, otherwise there may be danger, so you must be very careful.

•Precautions in use

caveat:

1) When an abnormal situation occurs during the operation of the instrument or the patient, the operator immediately removes the operating handle from the patient and presses the stop switch to stop the use of the instrument;

note:

2) During the operation of the instrument, it is necessary to continuously observe the condition of the beauty guest and the instrument, and pay attention to whether there is any abnormality at any time;

3) The instrument should be used correctly by a skilled operator, and no personnel other than the operator should use the instrument.

•Precautions after use

note:

- After the operation is over, put the control handle back into the control handle hanger.
- The probe should be cleaned after using the operating handle for the next use.

•Precautions for failure, storage and inspection

caveat:

- Do not use the instrument when abnormal conditions occur. In the unlikely event that there is a smell of smoke and burning and cannot be operated, if you continue to use it, there is a risk of fire and electric shock. Please quickly press the stop button, turn off the power, unplug the power plug, and immediately contact our company or dealer;
- Persons other than technicians must never repair or modify the instrument. Never open the inside of the instrument;

note:

- Instruments and components must be checked regularly;
- Do not use cleaning tools that damage the protective coating of the instrument, such as brushes, brooms, etc.;
- When the instrument is not used for more than 2 weeks, the instrument must be checked carefully before using it again.

•Warning and suggestive instructions

caveat:

When the high-frequency equipment and the instrument are connected to the same patient at the same time, the electrode position of the instrument may cause burns to the patient's skin tissue, and may also cause damage to the instrument. When using this instrument near shortwave or microwave equipment (1m), the output of the instrument may be unstable due to microwave interference.

Using electrodes close to the chest increases the risk of heart fibrillation.

b) Contraindications

1. Those with sensitive skin and severe acne
2. Cardiac rhythm device, implantable cardiac defibrillator or other electromagnetic implantable medical devices

3. Patients with severe diabetes, high blood pressure and hyperlipidemia
4. People with skin infections

3 How to install the instrument

a) Introduction to the instrument



b) Instrument installation instructions

(1) Unpack and check

① Carefully take out the instrument and accessories from the packaging box, and save the packaging materials for future transportation or storage. Check the accessories according to the packing list.

② Check if there is any mechanical damage.

note:

If you have any questions, immediately contact our sales department or agent.

(2) The instrument should be placed in a clean, ventilated room with a temperature of 5°C to 40°C and a humidity of no more than 80%, and avoid direct sunlight. And make sure that

the vent is not blocked, and make sure that there is at least a 5cm gap around the instrument. Remember not to place objects on the instrument, even if it is temporary, it is not allowed.

(3) According to the text and picture information prompts above the output interfaces on both sides of the instrument, insert the accessories into the corresponding interfaces. Then place the accessories neatly on the accessory rack.

C) Description of the back and side components of the instrument




D) When moving the instrument, please remove all the liquid on the Bottle A,B,C,D, otherwise, liquid will enter the inside of the instrument during moving, causing damage to the instrument.

E) When using the instrument, if the instrument sounds “beep beep”, please observing the interface for the wasted bottle. If it is twinkling, please pour out the wasting water and install the wasted bottle to the instrument after done pouring out.

4 Operation procedures and methods

4.1 Using for Aqua facial handle

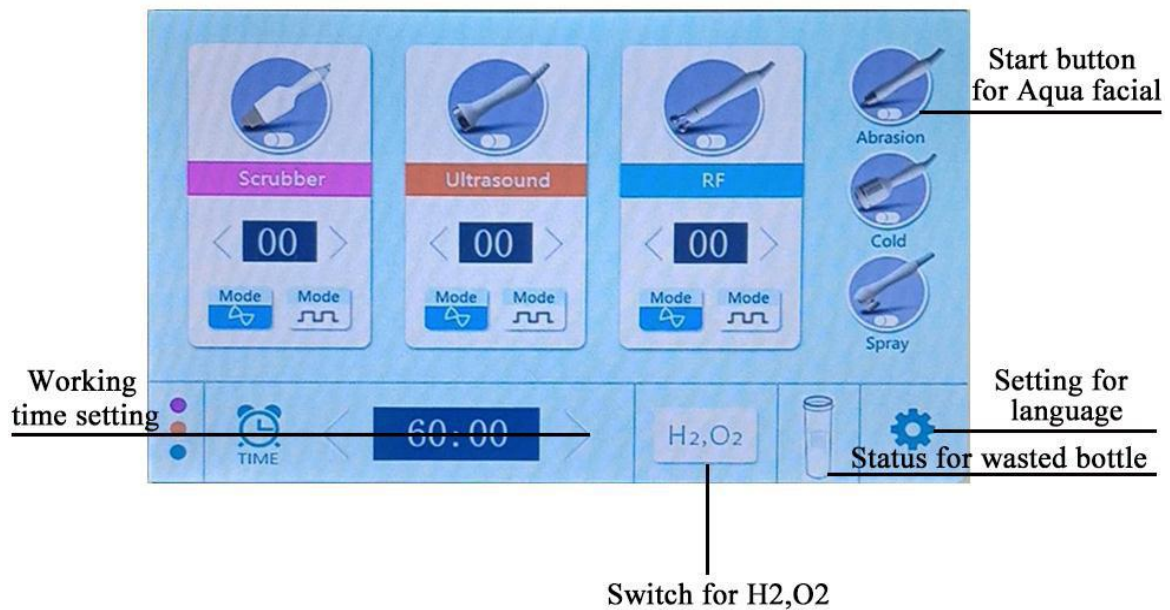

4.1.1 Cleaning the face by using facial cleanser.

Liquid skin care product for clean face, add it in A/B/C. If you have not it, use pure water or distilled water. And suggest to clean the face before treatment, make face without any cosmetic residue.

4.1.2 Choosing the aqua facial cleaning head according to the situation for the skin. Choosing the liquid (A,B,C) by rotating the knob or the H₂, O₂.


Rotate the button to select the water bottle: ABC for (Abrasion); When using the Spray handle, select D, if you want H₂, O₂, you must also click “H₂O₂” on screen.

4.1.3 Adjusting the flow level and pressure lever to start working, the whole cleaning procedure can be working at about 10 minutes.

Caution:

When using the H₂O₂ from the bottle D which on the back of the machine, first select the knob to the D position, and press the Button (H₂O₂), then please waiting for about 3 minutes, to make sure there are enough (H₂O₂) inside the bottle.


4.1.4 When done using the instrument, select the knob to position C to cleaning the aqua facial tube. First, selecting the flow level to the maximum intensity, and then selecting the pressure level to the maximum. Then turn on the machine, putting on the aqua facial head to the aqua facial probe. Then the tubes inside the machine will be cleaned automatically.

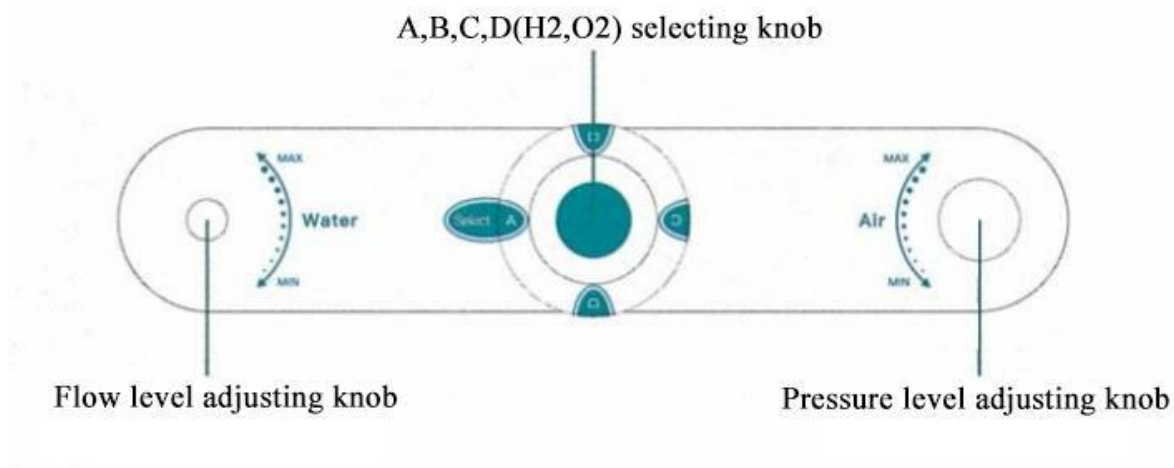

Caution:

When the liquids on the bottles are been using out, please input warm water to bottle A,B,C,D and cleaning the tubes by the methods for 4.1.4, to avoid blocking for the tubes.

4.2 Using for Spray handle

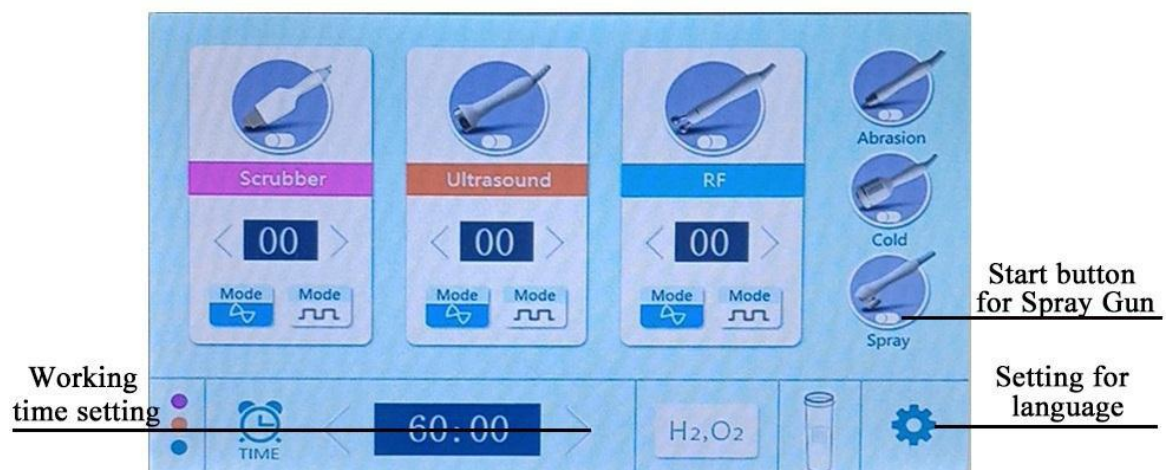


4.2.1 Cleaning the treatment area before using the handle.

4.2.2 Pour the diluted skin care product (essence of moisturizing or rejuvenating effect) into the container on the airbrush

4.2.3 Press the start button for the spray gun on the screen.

4.2.4 Press the button on the spray gun then the gun will start working and aim at the treatment area.

4.3 Using for Cold hammer

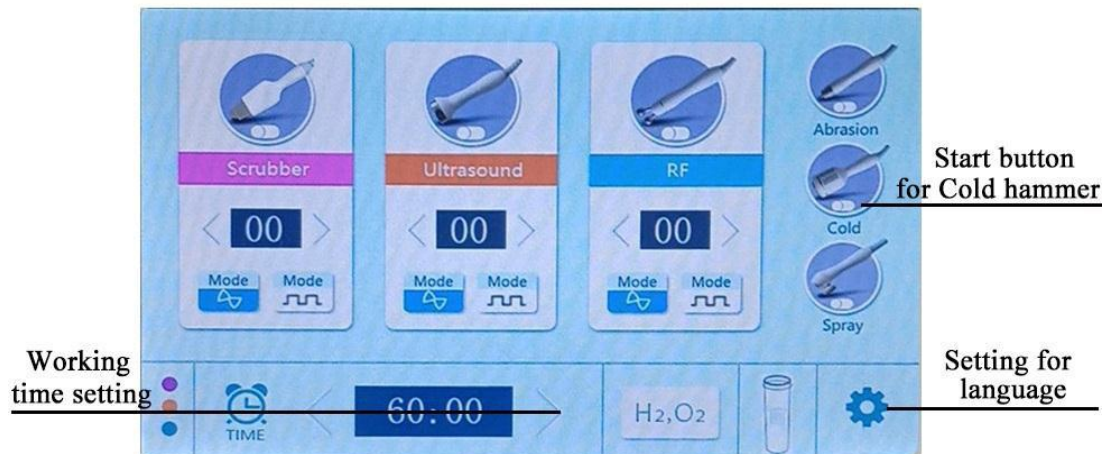

4.3.1 Cleaning the treatment area before using the cold hammer.

4.3.2 Press the start button on the screen.

4.3.3 Using the cold hammer on the treatment area.

Treatment on the face:

Start from ophryon and spread away, move upwards. As the photo shows.


(1) From lower jaw to ear.

(2) From lower jaw to nose.

(3) From nose to Temple.

(4) From inner corner to the temple,

do not get too close to the eyes and be tender.

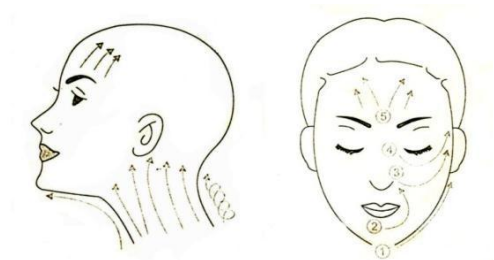

Suggest: Method of operation



4.4 Using for RF



4.4.1 Cleaning the treatment area before using the instrument.


4.4.2 Apply suitable gel on the treatment area.

4.4.3 Adjusting the working time, RF intensity and RF mode before using.

4.4.4 Using the RF handle stay close on the treatment area, and press the start button on the screen to start the RF function.

4.4.5 According to the facial line and neck, do circling from bottom to top to realize detox. And operation time for each part should be around 10-15 minutes.

4.5 Using for Ultrasonic

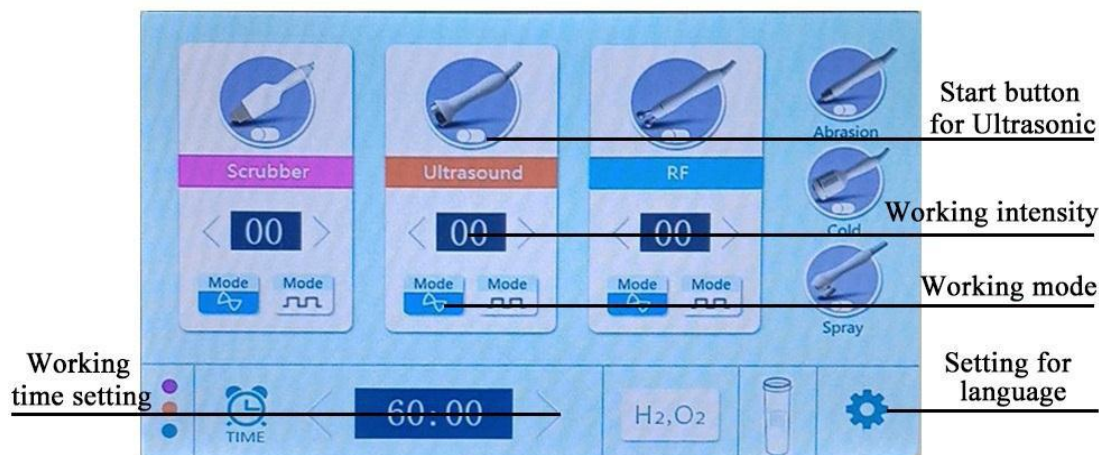

4.5.1 Cleaning the treatment area before using the treatment.

4.5.2 Apply suitable gel on the treatment area.

4.5.3 Adjusting the working time, ultrasonic intensity and ultrasonic mode before using.

4.5.4 Using the ultrasonic handle stay close on the treatment area, and press the start button on the screen to start the ultrasonic function.

4.5.5 According to the contour line and do circling for the treatment area at about 5-8 minutes.

4.6 Using for Skin scrubber



4.6.1 Cleaning the treatment area before using the treatment.

4.6.2 Adjusting the working time, intensity and mode before using.

4.6.3 Using the cleaning method to remove the dirt from the skin surface.


- ① Using the facial cleanser to clean the skin.
- ② Sterilizing the skin scrubber before using, and let the customer hold the sticker when using.
- ③ Applying enough water on the skin.
- ④ The blade is tilted at 45 degrees, make sure it is close to the skin, moving up and down at moderate speed.
- ⑤ The whole working time is 20 minutes.

4.6.4 Using the touching method to do lymphatic massage.

- ① Using the facial cleanser to clean the skin.
- ② Sterilizing the skin scrubber before using, and let the customer hold the sticker when using.
- ③ Applying enough skin care products on the skin.
- ④ Make the skin scrubber head flating with the skin and moving it according

to the texture of the muscles.

⑤ The whole working time is 15 minutes.

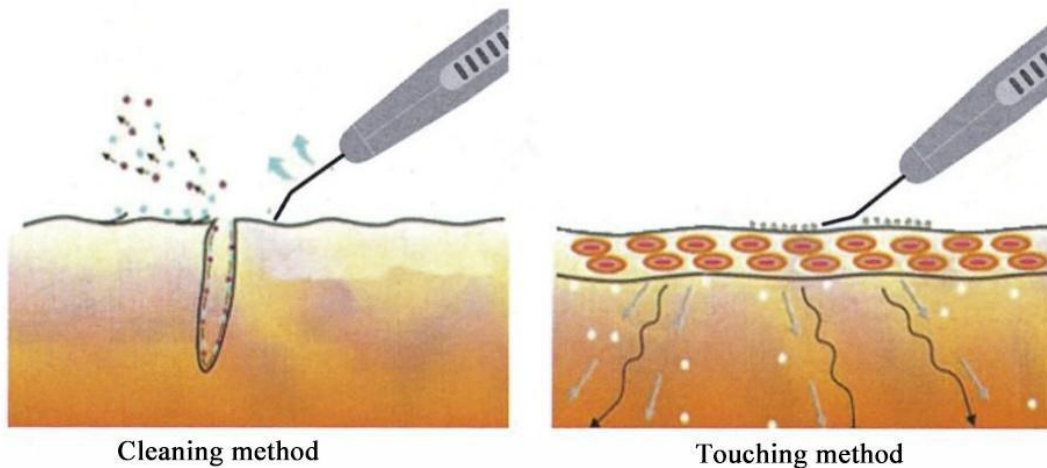

4.7 Notes

note:

- Do not use irritating skin care and cosmetics within 24 hours after treatment, and pay attention to sun protection
- Apply pure moisturizing mask for 1 week
- Thin skin stratum corneum or severe red blood streaks in the cheekbones, do not use water-based pens
- For those with thin skin stratum corneum or severe red blood streaks on the cheekbones, the cheekbones have been taken during the RF video operation, and the time should not be too long
- Severe allergies, severe acne cannot operate

caveat:

If the patient feels unwell, you can stop the output of the instrument by pressing the hand control to ensure patient safety.

5 Maintenance service

5.1 The instrument is under warranty for one year except the accessories and the consumables.

We would provide maintain service after the warranty period, but payable.

5.2 If you want to replenish or purchase the consumables and accessories, please contact the distributor for advice.

5.3 Breakdown cause by human as below is out of our warranty.

① Breakdown cause by dismounting or transforming the instrument by customer himself/herself.

② Breakdown cause by incorrect or unreasonable maintenance.

③ Breakdown cause by beating or dropping the instrument. ④

Breakdown cause by incorrect operation.

⑤ Damage cause by irresistible disaster like earthquake, fire disaster.

5.4 Welcome for any comment and suggestion.

5.5 When cleaning machines, please use dry soft towel to swipe the machine for the dust, and use wet soft towel to swipe the machine for the dirty.

a) **Accessories**

Name	Qty
Power line	1 pc
Skin Scrubber	1 pc
RF handle	1 pc
Ultrasonic probe	1 pc
Cold hammer	1pc
Hydro facial probe	1 pc
Hydro facial probe head	7 pcs
Spray gun	1 pc